

### WWV90PK

### Converting from a Table Top to a Pole Mount





For use by or on the order of a licensed veterinarian.



Attention, see operation manual for additional instructions for use.

# CDS9000 Pole Mount Kit Assembly Instructions to be used with WWV9010, WWV9011, or WWV9012

### **Table Top Base Removal Instructions**

1. Remove vaporizer from machine.

## WARNING! Do not tilt vaporizer. It is recommended to drain vaporizer before removing from machine.

2. Hold the machine head firmly and remove the four (4) ½" button head screws.

### **Pole Mount Assembly Instructions**

1. Assemble the mounting arm to the machine head using the  $\frac{1}{4}$ " nylon washers and the  $\frac{3}{4}$ " button head screws.

### NOTE: The mounting arm must be installed behind the frame of the machine head.

- 2. Tighten each button head screw firmly using a 5/32" Allen Wrench.
- 3. Insert the mounting arm with the machine head attached into the pole.
- 4. Align the threaded hole in the mounting arm with the hole at the back of the pole.
- 5. Install the  $1\frac{1}{2}$ " button head screw into the pole and thread into the mounting arm.

#### NOTE: Be careful not to cross thread.

- 6. Tighten the button head screw using a 5/32" Allen Wrench.
- 7. Install vaporizer.

### The WWV90PK CDS9000 Pole Kit includes:

PART #	DESCRIPTION	QTY
32076B1	1⁄4" Nylon Washer	2
32820B1	¾" Button Head Screw	2
32821B1	1½" Button Head Screw	1
32804B1	Mounting Arm	1
32028B6	Pole	1
32059B1	Chrome Base	1
32357B1	Caster	5
32411B2	5/32" Allen Wrench	1

 $\hbox{@2009 Smiths Medical Family of companies. All rights reserved.}\\$ 

SurgiVet and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol  $(\mathbb{R})$  indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.


Manufactured By:


\*\*\*

Smiths Medical PM, Inc. N7W22025 Johnson Drive Waukesha, Wisconsin 53186 Phone: 1-262-513-8500

Toll-Free: (U.S.A. only) 1-888-745-6562

Fax: 1-262-542-0718 www.surgivet.com

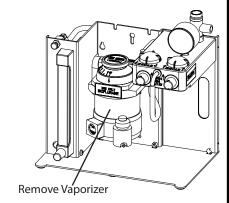

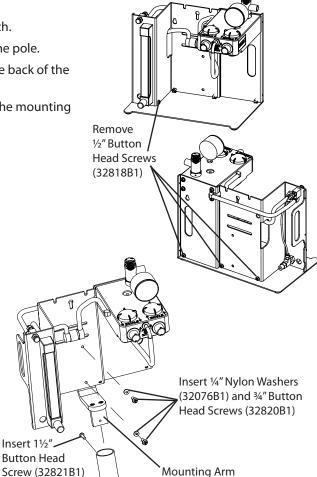

smiths medical