

Did you remember to check the species code on the last glucose test?

Eliminate the uncertainty by upgrading to the easier to use GlucoGauge monitor.



- ☑ Specifically calibrated for cats & dogs with an easy one button switch between species – no species coding required
- ☑ It's the only pet meter on the market with positive visual confirmation of the species being tested
- ☑ Includes species-specific results and verification during testing, in the results screen and in memory
- ☑ Only a small sample size is needed (.3uL) and get test results in 5 seconds
- ☑ Accurate & precise results: Relative difference of 3.4% for dogs and -.2% for cats compared to the leading brand relative difference of 8.4% for dogs and 9.5% for cats*

*Relative differences versus the Yellow Springs International Y2300

IMPORTANT NOTE: The GlucoGauge only uses GlucoGauge Test Strips. Also the GlucoGauge Test Strips will not work in other meters.



Helping to improve your clinic's profitability, without lowering your quality of care.

PVL00025-5/31



■ GlucoGauge Blood Glucose Test Kit

Henry Schein

GlucoGauge is an accurate, easy-to-use blood glucose meter for home use by pet owners. This meter is made specifically for dogs and cats, and provides blood glucose levels in seconds.

With the species code confirmed in the results and in memory, the GlucoGauge meter is easy-to-use both at home and in the vet clinic.

Directions for use: Simply load a lancet into the lancing device, twist-off the lancet top, turn on the Vet GlucoGauge and load a test strip into it, clean the sample site with an alcohol wipe. Let dry, and then prick the site with the lancing device, and touch the end of the test strip to the drop of blood from where the animal was pricked. The results are displayed within a couple of seconds.

The GlucoGauge Blood Glucose Monitoring System contains the following items:

- 1 GlucoGauge Blood Glucose Meter
- 1 GlucoGauge Owner's Manual
- 1 Vet GlucoGauge Quick Reference Guide
- 25 GlucoGauge Blood Glucose Test Strips
- 1 GlucoGauge Control Solution
- 1 Lancing Device
- 30 Lancets
- 1 Carrying Case
- 2 CR2032 Batteries

SKU#	MFR#	DESCRIPTION	UOM
056198	PT-100HS	Test Kits - Blood Glucose Vet Gluco-Gauge Kit	1 Kit

■ GlucoGauge Blood Glucose Test Strips

Henry Schein

GlucoGauge Blood Glucose Test Strips are validated for dogs and cats. To be used only with the GlucoGauge Blood Glucose Monitoring System. GlucoGauge Test Strips will not work in other meters.



SKU#	MFR#	DESCRIPTION	UOM	PKG
056199	PT-105HS	Test Kits - Blood Glucose Vet Gluco-Gauge Strip Box	50 Count	BOX

■ GlucoGauge Blood Glucose Test Control Solution

Henry Schein

Glucose control solution allows you to perform quality control tests in order to ensure that the GlucoGauge Blood Glucose Monitoring System is working properly and that you can rely on the test results. The control solution contains a fixed amount of glucose.



SKU#	MFR#	DESCRIPTION	UOM	PKG
056200	PT-116HS	Test Kits - Blood Glucose Vet Gluco-Gauge Control Solution	4mL	BOTTLE

■ GlucoGauge Blood Glucose Test Lancets

Henry Schein

Obtaining a blood sample is required for glucose monitors, and may be done by using a lancing device.



SKU#	MFR#	DESCRIPTION	UOM	PKG
056201	308HS	Test Kits - Blood Glucose Vet Gluco-Gauge Twist Top Lancets	100 Count	BOX

■ GlucoGauge Blood Glucose Lancing Device

Henry Schein

The Lancet Device is used to collect blood samples from your diabetic pet in order to measure his/her blood glucose (sugar) levels. This device is a reliable way to acquire a drop of capillary blood in order to run blood glucose tests as directed by your veterinarian. The Lancet Device can be used with the GlucoGauge Blood Glucose Monitoring System.



SKU#	MFR#	DESCRIPTION	UOM
056202	328HS	Test Kits - Blood Glucose Vet Gluco-Gauge Lancing Device	1 Device

PVL00025-5/31



Helping to improve your clinic's profitability, without lowering your quality of care.

